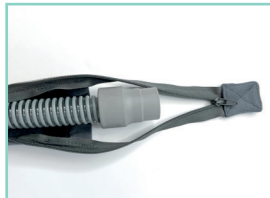
## FUNDA DE MANGUERA OXYHERO CPAP

ES

**Color:** antracita

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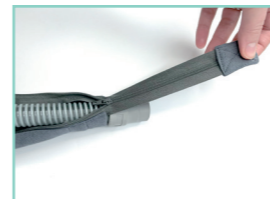
# oxyhero

CPAP hose cover

## OXYHERO CPAP HOSE COVER [ANTHRACITE]

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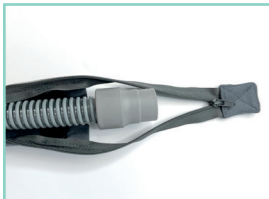
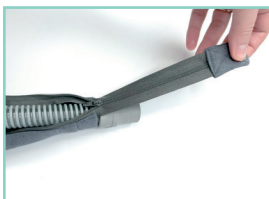
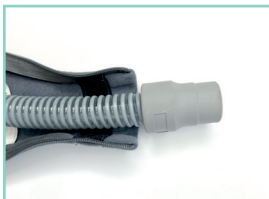
EN

**Colour:** anthracite

**Textile composition:** 100% cotton (from organic agriculture)

**Application:** Open the zip fastening on the hose cover.

Guide the end of the hose that is going to be connected to the CPAP device through the open end of the zip fastening (bottom end of the zip).

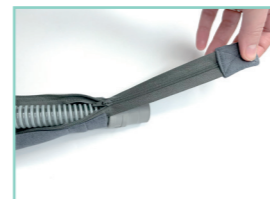
Before closing the zip fastening, make sure that the connection sleeves protrude beyond the end of the hose cover by approx. 1 cm on both sides.

Close the zip fastening.

Connect the CPAP hose as follows: connect the connection sleeve where the bottom end of the zip is (= where the zip fastening is protruding beyond the end of the hose cover by approx. 5 cm) to your CPAP device. Connect the opposite side (where the top end of the zip is, i.e. where the end of the zip fastening sits flush with the hose cover) to your CPAP mask.

**Please note:** Wash on a normal wash cycle at a temperature of up to 40°C. The hose cover cannot be bleached, i.e. it should only be washed with colour detergents or mild detergents. Dry on a delicate programme in the tumble dryer.

**Warnings:** Do not use in conjunction with heated CPAP hoses (risk of fire!).

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