# oxyhero

push

**Hinweise/Information/Notes/Indicazioni/Notas/  
Informatie/Huomioitavaa**

**Lieferumfang/Delivery includes/Contenu de la livraison/Ambito di consegna/  
Alcance de la entrega/Omvang van de levering/Pakkauksen sisältö**

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## **User information leaflet • ENGLISH**

### General information

oxyhero push is a nasal dilator that gently opens up the nostrils, allowing you to breathe more easily through

Video instructions  
for correct use

your nose during sleep and exercise.

During exercise, improved nasal breathing can result in up to 30% greater breathing capacity (respiratory volume) – particularly in some mainstream endurance sports such as running or cycling.

During sleep, being able to breathe freely through the nose results in less air turbulence forming in the upper airways. This results in less vibration of the throat tissue and in turn less snoring.

Moreover, nasal breathing enhances your body's regeneration process, as it increases blood oxygen saturation by up to 15%.

### Instructions for use for the trial pack (Size S/M/L)

1 First of all, clean your nasal passages. A neti pot (such as NoseBuddy) is a good way to do this.

2 We recommend standing in front of a mirror the first time you insert oxyhero push. This makes insertion easier.

3 Remove oxyhero push size M from the packaging. The nasal dilator consists of two nasal inserts joined together by a connector bar. On the lower rim of each insert there are also two soft, slender 'wings'. These wings provide a better hold in the nose, as they create an 'anchoring effect'. If you do not need the wings or find them uncomfortable, you can simply shorten them or cut them off with a pair of scissors.

4 To insert oxyhero push, hold the product so that the nasal inserts face upwards (meaning the horseshoe-shaped connector bar points downwards). Now use your fingers to hold down the wings

of each nasal insert, meaning the wings lay flat against each insert and point upwards, as do the inserts. Now use light pressure to push the oxyhero push into your nose. The nasal dilator should be a comfortably snug fit – the wings give it extra support as they gently spread open inside the nose.

5 If the nasal dilator sits too loosely, please repeat **step 4** with oxyhero push in size L. If M is too tight, please use size S. To make insertion easier, you can also moisten the nasal dilator with water beforehand. If you experience skin irritation, you may also wish to use a nasal ointment.

6 If you have a very small or narrow nose, you may not be able to insert oxyhero push quite so deep.

The nasal dilator should not cause any pain when you wear it. Naturally, you may expect to feel a sensation of a foreign body in your nose to begin with, but this will generally disappear after a short period of acclimatisation.

7 To remove oxyhero push, just pull it gently from the nostrils.

#### **Directions for use for single-size product packs (up to 3 units of a single size in one pack)**

1 First of all, clean your nasal passages. A neti pot (such as NoseBuddy) is a good way to do this.

2 We recommend standing in front of a mirror the first time you insert oxyhero push. This makes insertion easier.

3 Remove oxyhero push from the packaging. The nasal dilator consists of two nasal inserts joined together by a connector bar. On the lower rim of each insert there are also two soft, slender ‘wings’. These wings provide a better hold in the nose, as they create an ‘anchoring effect’. If you do not need the wings or find them uncomfortable, you can simply shorten them or cut them off with a pair of scissors.

4 To insert oxyhero push, hold the product so that the nasal inserts face upwards (meaning the horseshoe-shaped connector bar points downwards). Now use your fingers to hold down the wings of each nasal insert, meaning the wings lay flat against each insert and point upwards, as do the inserts. Now use light pressure to push the oxyhero push into your nose. The

nasal dilator should be a comfortably snug fit – the wings give it extra support as they gently spread open inside the nose.

**Caution:** If the nasal dilator sits too loosely, it may be that you need to select a different size. For the first use, we recommend a trial pack with mixed sizes (S/M/L). The nasal dilator should not cause any pain when you wear it. Naturally, you may expect to feel a sensation of a foreign body in your nose to begin with, but this will generally disappear after a short period of acclimatisation.

5 To remove oxyhero push, just pull it gently from the nostrils.

#### **Cleaning instructions**

oxyhero push should be cleaned thoroughly with warm water after each use.

After that, please dry with a clean towel. Do not use cleaning agents that are abrasive or alkaline, as these corrode the material and should not come into contact with the sensitive mucous membranes of the nose. Store oxyhero push in a clean, dry place.

### Important information

Do not use oxyhero push if you have issues with insertion due to an asymmetrical nose or if you are known to suffer with moderately or severely impaired nasal breathing. In addition, the nasal dilator is not suitable for the treatment of obstructive sleep apnoea. In these instances, please seek advice from your ENT doctor or a sleep specialist before use.

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