

Löwenstein Medical

Special Descaling Concentrate



Professional Descaler with Amidosulfonic Acid

- Up to 3x more economical
- Up to 6x stronger
- Up to 8x faster

... than conventional citric acid-based products

Content:

500 ml

Ingredients according to EC 648/2004 VO Detergents:

< 5 % phosphates

Instructions for use:

Dilute concentrate with water at a ratio of 1:5 to 1:10, depending on limescale level. Follow manufacturer's descaling recommendations. Rinse thoroughly with clean water afterwards.

Important notes:

Always follow the descaling instructions provided by your appliance manufacturer. Do not allow contact with acid-sensitive surfaces (e.g. enamel, aluminium, marble). Always test product compatibility on an inconspicuous area before use. No liability will be accepted for damage resulting from improper use. Protect from frost.

Product safety information:

H315 Causes skin irritation, H319 Causes severe eye irritation.

P102 Keep out of the reach of children. P280 Wear protective gloves/protective clothing/eye protection/face protection. P302+P352 IF IN SKIN CONTACT: Wash with plenty of water. P332+P313 In case of skin irritation: Seek medical advice/attention. P305+P351+P338 IN THE CASE OF EYE CONTACT: Rinse gently with water for several minutes. Remove existing contact lenses if possible. Continue rinsing. P337+P313 If irritation persists: seek medical advice/attention.

Manufacturer: HOTREGA GmbH, Lorenz-Weber-Straße 2, 36364 Bad Salzschlirf,
info@hotrega.de